Cervipol soft 500

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specifications

The Cervipol Soft 500 is made of foam-material and covered with cotton. It is eudermic and provides high wearing comfort. The polyethylene reinforcement grants stability for the cervical spine.

Indication

- distortion of cervical spine (including whiplash)
- confusion of cervical spine
- severe pains
- degenerative deformans
- inflammation
- nape-shoulder-arm-syndrome
- postoperative

Instructions

Place the cut-out under the chin and close with velcro in the back of the neck.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Cervipol soft 500.

Washing instruction (🧼 🚰 🚰 🚰 🚰)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid wringing as well as direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Colour

beige