Cerviplus 100

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The cervical collar Cerviplas 100 is made from antibacterial and moisture repellent Plastazote. It is easy to take care off and due to the perfect conformity to the anatomy of the neck it provides high wearing comfort. The Cerviplas 100 stabilizes and fixes the cervical spine. Perforations in the Plastazote grant optimal circulation of air. Therefore the build-up of heat will be avoided.

Indication

- radicular syndrom of cervical spine
- distortion of cervical spine incl. whiplash injury
- protruding vertebral disk and prolapse
- nape-shoulder-arm syndrome
- degenerative deformans
- torticollis
- postoperative

Instructions

Place the cut-out under the chin and close with velcro in the back.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. To ensure the correct fitting the Cerviplas 100 has to be applied by your physician or orthopaedic technician for the first use. The Cerviplas 100 can be customised thermally and/or mechanically to the anatomy of the patient.

Washing instruction (☞ ☒ ☒ ☒ ☒)

The cervical collar may be cleaned with a sponge and mild detergent. For drying use a towel. Do not dry in direct heat (e.g. sunlight, radiator, drier, etc.). Do not bleach or use washing sodas. Do not iron or dry clean.

Colour

skin-coloured