Re-Lax Kissen

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

Three different sizes, the thermoactive features in relation with the visco elastic properties result a perfect conformity. So you really have a high quality pillow, which relieves and relax the head and the shoulder musculature at a right anatomical position of the cervical vertebra. These properties result an ideal distribution of the weight and in that way result a pressure reduction of the area of use, reduction of the risk of decubitus because the pressure relief does not clog the blood circulation.

Indications

- neck pains, headache, backache, migraine
- relief of cervical vertebra
- tenseness
- sleeplessness caused by wrong posture
- for relax

Notice

Pains, circulation problems and tenseness in the neck, back and shoulder area are caused by a wrong posture of the body. The ergonomic "Re-Lax Kissen" of Orpedo relieves the paths, musculature and the vertebra in the neck area. Due to the use and habituation in to the pillow as well as due to the thermoelastic features of the material you will feel after a short time how relaxed and rested you are. Please don't fold the pillow or give it with force another shape because the material could be damaged.

Washing instruction ()

The pillow case can be removed easily and be hand washed at 30°C. Please turn it in side out. At this opportunity put the pillow out to air in order to keep a low-level humidity. When fist unpacked the product has probably over a short time a harmless material smell. If necessary the foam material can be wiped off with a damp cloth.

Guarantee

For the Re-Lax pillow of Orpedo there is a guarantee time of 3 years (from the purchase day). In case of claims under the guarantee you have to present a delivery note or an invoice as document of your purchase. The guarantee covers damages or changes of the material features which cause a visible indentation (more than 2 cm) in the material. The guaran tee does not include normal changes in hardness or in material features which don't affect the pressure-relieving properties. The guarantee can be demanded only if the pillow has been used in accordance with the product's instruction.

Sizes

Small (52x31x9/7cm)	Medium (52x31x10,5/7cm)	Large (52x31x12/7cm)
------------------------	----------------------------	-------------------------