Clavivit 380 Claviculabandage

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

Clavivit 380 for the immobilization of the acromioclavicular articulation, reduction of the shoulder resp. clavícula. The pull piece to reduce the shoulder is individually adjustable. In case of injuries in the acromioclavicular articulation resp. in case of fracture of the clavícula, the pull piece of the arm loops causes a repositioning of the clavícula. An excellent immobilization of the shoulder girdle is achieved. By means of the shoulder axilla pads individually movable, the pull piece in direction of the opposite side and the wide breast belt to distribute the pressure, an optimal burden distribution is achieved. The optimal position of the shoulder girdle with the help of the Clavivit bandage of Orpedo offers a wide spectrum of utilization in case of the following indications.

Indications

- Injury of the musculotendinous cuff, insufficiency of capsule/ligament apparatus after trauma
- Immobilization of acromioclavicular articulation after trauma
- for postoperative or conservative care of fractures of the clavícula
- for post-traumatic or postoperative immobilization
- postural defects
- distortion, luxation
- traumatic lesions
- degenerative alterations

Instructions

The bandage consists of two parts. Each with arm loop, movable shoulder axilla pads and wide breast belt. In the connecting area between arm loop and breast belt strengthening plastic parts are incorporated. Lead the arms through the arm loops of bandage (seam on the inside, plastic strengthening on the outside). Close the breast belts over the stomach (Orpedo sticker on the outside). Most of the clothing material react sensitively to Velcro fasteners. That is why we recommend to avoid any contact between the Velcro fasteners and clothings because this might cause threads at the closings.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the support. In case of numbness or coldness loosen the support or take it off at once.

Washing instruction ()

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

white