## Neoflex Epicon 450

### Kostenlose Service Hotline

0800-ORPEDO.  
0800-6773361

**Orpedo**  
Orthopedics Company KG

Knesebeckstraße 54  
D-10719 Berlin

**Tel** +49-30-88 00 77 10  
**Fax** +49-30-88 00 77 14

Postfach 15 05 44  
D-10667 Berlin

**Web** [www.orpedo.com](http://www.orpedo.com)  
**Mail** [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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## INSTRUCTION FOR USE

### Product description

The **Neoflex Epicon 450** has a silicone pad underneath the buckle at the inner side of the support. The pad induces the compression on the muscle tendon. The Neoflex Epicon 450 supports the elbow, eases pain and reduces the risk of injuries. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic. Neoprene provides high wearing comfort.

### Indications

- enthesopathy in the area of the elbow (tennis or golf elbow)
- irritations at the muscle tendon of the epicondyle
- prophylaxis during sport activities

### Instructions

Wrap the elbow support round the arm, the silicone pad should be placed slightly beneath the elbow at the inner or outer side of the joint depending on the location of discomfort to relief the muscle tendon. For the application of this product you have to follow its description and intention.

### Notice

Please contact your physician or pharmacist in any of the following cases: open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Epicon 450. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

### Washing instruction ( )

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

### General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

### Color

black