Neoflex Carpivit 410

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

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INSTRUCTION FOR USE

Specification

The wrist support Neoflex Carpivit 410 restrains the warmth and gives firm support to the wrist. The thumb loop prevents the bandage from shifting. The velcro closure allows an individual adjustment. The Neoflex Carpivit 410 grants height wearing comfort while working or during sport activities. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic. Neoprene provides high wearing comfort.

Indication

- prophylaxis during sport activities
- slight degenerative and rheumatic diseases or enthesopathy giving firm support to the wrist
- pain by carpal tunnel syndrome
- overuse injuries
- posttraumatic
- tenosynovitis

Instructions

Push the thumb through the loop and wrap the Neoflex Carpivit 410 round the wrist and close with Velcro.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Sever injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Carpivit 410. In case of numbness or coldness loosen the support or take it of at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (☞ ☒ ☒ ☒ ☒)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Color

black