Pollicivit 420

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

Pollicivif 420 is designed in an exactly, to the anatomy of the hand and the thumb fitting way. On the inside, there is an aluminium brace that is anatomically preformed and can be adjusted in a form-fitting manner according to the individual circumstances. Velcro fasteners provide an individual adjustment possibility, an easy handling and a perfect fit of the aid. The thumb orthosis can be used at the right and at the left hand. The aluminium brace that can be easily and completely adjusted in adjacent manner, in connection with a plastic strengthening and the three different sizes make the orthosis be an optimal aid in case of the following diagnosis.

Indications

- degenerative disease
- stabilization of the saddle articulation of the thumb (principle articulation of the thumb)
- chronic, post-traumatic or postoperative irritation in the area of the saddle articulation of the thumb
- postoperative or conservative care
- prevention against overburden

Instructions

Put the thumb into the orthosis resp. the orthosis around the hand. Check the formfitting position of the aluminium brace and adjust it according to the individual circumstances. For a perfect position of the thumb, pay attention that the aluminium brace is not hardly bent but fits form-fitting, in a round way and reaching into the corners. Close the Velcro fasteners firmly.

Notice

The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the support. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Material

Nylon
Polyurethan sponge
Polyethylene
Aluminium

Washing instruction (🧼🔥🚫🧴🧴)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

black