Lumbovit 200

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

Because of its anatomical shape the **Lumbovit 200** supports the lumbar vertebrae area. There are four plastic supports with sizes S, M, L and six plastic supports with size XL right and left from the middle part to give optimal support to the back. Two additional straps had to be brought to the front and placed beneath the abdomen to lift and support the abdomen. This causes a shift in the balance to the centre of the body. The support can be easily closed and adjusted by velcro closures.

Indications

- hardening
- malposition
- degenerative deformans (spondylitis, ischialgia, low back pain, spondylolisthesis)
- postoperative
- pain
- inflammation
- chronic or acute pains at the lower spine

Instructions

Wrap the support around the waist and close the velcros in the front. Bring the straps from the back to the front and attach them to the front part beneath the abdomen to ensure a lifting of the abdomen. For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Lumbovit 200. Do not use ointment while wearing the support.

Washing instruction ()

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Spare parts

Order number 908-904 plastic support

Order number 908-905 steel support

The steel supports can be customised to the anatomy of the patient.

Color

white