Malleovit Comp 660 sw

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

The ankle bandage **Malleovit Comp 660 sw**, consisting of dual pull elastic material, is anatomically knitted. The knitted comfort zone in the instep area and lateral insole pads to lay the malleolus stimulate the circulation and support the normal motions. The breathable material is particularly kind to the skin, avoids hypothermia and ensures a comfortable wearing. The entire effect of Malleovit Comp 660 sw is achieved in motion. The compressing characteristic of the bandage causes a muscular support and so stimulates the massage of the soft parts, an improved lymphatic drainage and detumescence.

Indications

- irritations after operation, trauma, overburden of tendons, ligaments (lateral resp. medial) and/or muscular system
- chronic pain in the articulation of the ankle caused by degenerative or rheumatic disease (arthrosis)
- postoperative treatment
- slightly injuries in the articulation of the ankle (contusions, disortions, distension)

Instructions

Pull the Malleovit Comp 660 sw like a knee-high over the put on aid that ist included in the consignment. Pay attention to the correct fit of the pads under the inside and outside ankle points. Pull the put on aid forward out of the bandage. Use the product according to the indications and the purpose.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Malleovit Comp 660 sw. In case of numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist to check the fit and adjustment.

Washing instruction ()

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

black, grey