Air Support 600

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Material

Polyacetal
Nylon
TPU-Material (Polyurethan)
Polyester-Schaum

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INSTRUCTION FOR USE

Product description

The air padded system with two rigid plastic moulds provides optimal positioning of the ankle joint. It provides ligament support and can be individually adjusted by Velcro® tapes. Sideways motion is only slightly possible and rolling motions are only restricted possible. The **Air-Support 600** is ideal for the prophylaxis during sport activities and for the rehabilitation of sport injuries. The textile covering surface of the air chambers provides high wearing comfort reducing the sweat forming.

Indications

- Fibula ligament injuries
- Distortion
- Postoperative
- Inflammation
- Lateral rotation trauma
- Contusion
- Degenerative deformations

Instructions

Open the Velcro® tapes and pull apart the plastic moulds. Put the foot into the ankle support and place the plastic moulds of the Air-Support 600 on the right and left side corresponding to the label int. and ext. for each side. The ankles are now on the hole of the moulds. Now close the Velcro® tapes circularly. Air chambers can be filled with air individually with provided hang pump through the valve.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complications. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Air-Support 600. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first. We suggest wearing a sock beneath the support.

Material

Polyacetal
Nylon
TPU-Material (Polyurethan)
Polyester

Washing instruction ()

Clean the product with a wet cloth (do not use acrid agents). Please protect the air chambers against sharp objects.

Color

black

Size

universal