Calcavit 720 / 730

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Postfach 15 05 44
D-10667 Berlin

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Product description

Calcavit is a heel pad made of silicone, because of its special viscoelastic structure (composition of the silicone) it is unique concerning the bump absorption. That is why bumps on the articulation of the ankle, knee, hip and spinal column which are caused by walking/running are reduced to a minimum. The result is a considerable relief of the heel, the articulations, ligaments, muscles and tendons. The Calcavit heel pads are non-slip and can be worn in normal shoes, trainers and working shoes.

Indications

- to avoid pressure centres at the calcaneal spur
- attrition (arthrosis) of the articulation of ankle, knee and hip
- ileosacral arthrosis and arthrosis of facet articulation
- inflammation of a bursa
- to relieve artificial articulations (endoprothesis)
- pain at the Achilles tendon (achillodynia)
- in case of differences of leg length up to ca. 10 mm (balancing)
- after calcaneal fracture

Contraindication

Use and wear the heel pad only after having consulted your doctor in the following cases:

- skin disease/pain in the treated body area, especially in case of inflammation symptoms
- scar fissure
- tumescence
- reddening
- hyperthermia

Instructions

Put the Calcavit heel pad into the shoe so that it is form-fitting with the heel area of the shoe. Pay attention that the heel cap of the shoe you use is high enough because otherwise slipping effects might occur. Please wear knee-highs made of cotton.

Side effects

Sweaty feet may occur. The skin flora of the feet can regenerate by wearing knee-highs made of cotton and by daily, complete mechanical cleaning of the feet (brush etc.). Normally, after a few days the hidropoiesis already is reduced. Please ask your dermatologist or pharmacist.

Washing instruction

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

transparent