Carpivit-Therm 110

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The wrist support Carpovit-Therm 110 made of soft eudermic and elastic material protects the wrist against cold and draft. It is optimal when working outdoor or e. g. in cold storage rooms. It is easy to apply and can be worn by allergic persons since no angora or lambs wool was used.

The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic Neoprene provides high wearing comfort.

Indication

- overuse injury
- rheumatic
- trauma
- distortion

Material composition

polyacrylic: 80 %

spandex: 13 %

polyester: 7 %

Instructions

Wrap the Carpovit-Therm 110 round the wrist and close with velcro.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: sever injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Carpovit-Therm 110. In case of numbness or coldness loosen the support or take it of at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (🧼 🚰 🧴 🧺 🧻)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

wool white