## Genuvit-Therm Gel 140

### Kostenlose Service Hotline

0800-ORPEDO.  
0800-6773361

**Orpedo**  
Orthopedics Company KG

Knesebeckstraße 54  
D-10719 Berlin

Postfach 15 05 44  
D-10667 Berlin

**Tel** +49-30-88 00 77 10  
**Fax** +49-30-88 00 77 14

**Web** [www.orpedo.com](http://www.orpedo.com)  
**Mail** [info@orpedo.com](mailto:info@orpedo.com)

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## INSTRUCTION FOR USE

### Specification

The knee warmer Genuvit-Therm Gel 140 made of soft, elastic and eudermic material offers miscellaneous appliances in thermotherapy. Even without acute disorders the neck warmer grants an agreeable and cosy feeling especially during the cold months. The gel pad can be used without knee warmer for cryotherapy on other body parts as well.

### Indication thermotherapy

- rheumatic Diseases

### Indication cryotherapy

- contusion
- sprain
- distorsion
- good for stiffness after exertion, toothache, nosebleed, hematoma, migraine, fever

### Ideal for allergic persons

Since no lamb's wool or angora is used.

### Instructions

The knee warmer can be applied variable as desired and is closed with the broad velcro closure. Due to the firm hold the gel pad of the Genuvit-Therm Gel 140 remains in the afflicted area while in motion.

*For the application of this product you have to follow its description and intention.*

### Warning

Heat the gel pad 10 minutes by 60° C - 80° C. Or 7 minutes in the microwave oven by 150 W.

### Cooling

Freeze gel pad 2 hours in the freezing compartment.

### Contraindication by cryotherapy

Do not use cold compresses with high blood pressure.

### Important, read carefully

- remove gel pad prior to heating
- be careful when inserting hot gel pad to the back warmer (danger of burning)
- do not use damaged gel pads
- if the gel contacts closing wash the affected cloth with clear water
- do not hand to children without supervision of an adult, inappropriate handling of the gel pad may cause burnings as well as leaking gel may cause damages to health

### Notice

Please contact your physician or pharmacist in any of the following cases: sever injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit-Therm Gel 140.

### Washing instruction (☞ ☒ ☒ ☒ ☒)

After removal of gel pack the support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

### Color

wool white