## Genuvit-Therm 500

### Kostenlose Service Hotline

0800-ORPEDO.  
0800-6773361

**Orpedo**  
Orthopedics Company KG

Knesebeckstraße 54  
D-10719 Berlin

Postfach 15 05 44  
D-10667 Berlin

**Tel** +49-30-88 00 77 10  
**Fax** +49-30-88 00 77 14

**Web** [www.orpedo.com](http://www.orpedo.com)  
**Mail** [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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## INSTRUCTION FOR USE

### Specification

The knee warmer Genuvit-Therm 500 made of soft, elastic and eurdemic material protects against coldness and draft. The agreeable warmth relieves pain in the knee area. Due to the soft material and the good fitting the Genuvit-Therm 500 grants mobility and edges do not incise.

### Indication

- slight knee pain caused by injuries or overuse or arthrosis
- rheumatic diseases
- gonalgia
- warming the knee

### Material composition

Polyacryl: 80 %  
Spandex: 13 %  
Polyester: 7 %

### Ideal for allergic persons

Since no lamb's wool or angora is used.

### Instructions

Pull the knee warmer the over afflicted part.  
*For the application of this product you have to follow its description and intention.*

### Notice

Please contact your physician or pharmacist in any of the following cases: sever injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit-Therm 500.

### Washing instruction (🧼 🚰 🧴 🧺 🧻)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

### Color

wool white