Neoflex Lumbovit 300

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The Neoflex Lumbovit 300 made of high-quality neoprene supports the lower back and relieves pain and prevents further ailments. The support spends soothing therapeutic warmth. The two steel supports and two plastic supports with size S and four steel supports and two plastic supports with sizes M and L in the back area provide, in connection with the pad, optimal support to the lower back. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic Neoprene provides high wearing comfort.

Indication

- lumbago and ischialgia
- sensitivity to cold
- relief of the spinal column
- rheumatic pains
- conservative and postoperative with
- ventral abdominal wall hernia

Instructions

Attach the pad to the velcro tape inside the back of the support. Wrap the support around the waist and close with buckles.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. Since the supports have to be customised the first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Lumbovit 300. The supports of the Neoflex Lumbovit 300 can be customised mechanically to the anatomy of the patient. Do not use ointment while wearing the support.

Recommendation

It maybe advisable to wear a T-Shirt beneath the support.

Washing instruction (☑ ☒ ☓ ☔ ☕ ☖)

After removal of the stays the support may be washed at 30° C in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Spare parts

Order number 908-904 plastic support

Order number 908-905 steel support

The steel supports can be customised to the anatomy of the patient.

Color

black