## Neoflex Genuvit 340

### Kostenlose Service Hotline

0800-ORPEDO.  
0800-6773361

**Orpedo**  
Orthopedics Company KG

Knesebeckstraße 54  
D-10719 Berlin

Postfach 15 05 44  
D-10667 Berlin

**Tel** +49-30-88 00 77 10  
**Fax** +49-30-88 00 77 14

**Web** [www.orpedo.com](http://www.orpedo.com)  
**Mail** [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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## INSTRUCTION FOR USE

### Specification

The Neoprene knee support Neoflex Genuvit 340 with two flexible stays provides a secure support for the knee joint due to its anatomical shape and good fitting. The open patella area is surrounded by a silicone ring to stabilize the kneecap.

### Indication

- degenerative or rheumatic diseases also with medial and lateral instability
- postoperative
- slight knee injuries (distortion, contusion, distention)
- retropatellar chondropathy
- to stabilize the patella in case of patellar dislocation

### Instructions

Pull the support over the afflicted knee and place the reinforced seam at the back of the leg. The silicone ring should enclose the kneecap.  
*For the application of this product you have to follow its description and intention.*

### Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Genuvit 340. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

### Washing instruction (☞ ☒ ☒ ☒ ☒)

After removal of the stays the support may be washed at 30° C in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

### General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

### Color

black