Neoflex Manuvit 430

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The wrist is bedded with the Neoflex Manuvit 430 in an optimal posture. The possibility of movement is restricted by a prepared removable aluminium splint. Therefore the healing process of injuries and inflammation will be much faster. The support is easy to attach and the velcro closure allows individual adjustment of compression. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic. Neoprene provides high wearing comfort.

Indication

- carpal tunnel syndrome
- tenosynovitis
- support after removal of cast
- inflammation (arthritis)
- degenerative deformations (arthrosis)
- trauma (contusion, distortion, luxation, impaction)
- postoperative
- conditions after open wounds

Instructions

Open the velcos as far as possible and place the hand within the support. Close velcos.

The aluminium splint can be adjusted by an specialist.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complications. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Carpivit 410. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (☞ ☒ ☒ ☒ ☒)

After removal of the splint the support may be washed at 30° C in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Color

black