Neoflex Epicon 480

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The elbow support fits perfectly to the anatomy of the arm. In the area of the epicondyles a velcro strap is attached. Under the velcro strap there are two moveable neopren pads on velcro tape. The neopren pads induce a compression to relieve the muscle tendon. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic. Neoprene provides high wearing comfort.

Indication

- enthesopathy in the area of the elbow (tennis or golf elbow)
- following trauma and slight injuries
- bursitis
- postoperative and for after-treatment following serious injuries
- degenerative deformans of the elbow joint
- overuse pain

Instructions

Pull the support over your elbow and place the reinforced seam at the inner side of your arm. The neopren pads should be placed. Close by tearing the velcro through the buckle first and then backward. Attach tape to its counterpart.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Epicon 480. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (☞ ☒ ☓ ☒ ☒)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Color

black