Neoflex Malleovit 530

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The ankle support Neoflex Malleovit 530 made of Neoprene gives secure support to the ankle. For an increase of compression lateral silicone pads provides a secure positioning of the ankle. The Neoflex Malleovit 530 grants a support of ligaments due to the optimal positioning of the ankle joint. Heel and toe area are open. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic Neoprene provides high wearing comfort.

Indication

- distorsion of ankle joint with slight lateral and medial Ligament problems caused by ligament lesion
- contusion
- pain condition
- arthrosis
- after removal of plaster cast

Instructions

Pull the support over the afflicted ankle like a sock. Take care that the pads are positioned directly underneath the in- and outside knuckle.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Malleovit 530. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (☞ ☒ ☒ ☒ ☒ ☒)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Color

black