ParadiSol 700

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

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INSTRUCTION FOR USE

Specification

The maternity belt ParadiSol 700 supports the belly, shifts the weight and relieves the spine. It is made from eudermic fabric on which the belt can be closed variable. Therefore the maternity belt can be worn, in most cases, throughout the whole pregnancy. During the complete pregnancy an optimal support is granted and the freedom of movement is increased. After the birth of child the back formation of the tissue is supported.

Indication

- support for the spine
- prophylaxis by herniated nucleus pulposus

Instructions

Place the broad front part beneath the abdomen and close with velcro closure in the back.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the ParadiSol 700. Do not wear the support while sleeping. Do not use ointment while wearing the support.

Washing instruction (🧼 🚫 🧴 🧴 🧴)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

white