ParadiSol 800

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The maternity belt ParadiSol 800 supports the belly to prevent backache. Due to its specific cut the maternity belt holts up the abdominal cavity and therefore relieves the spinal column. During the advance of pregnancy well being and movement are ensured. Due to the lateral straps the tightness of the belt can be adjusted individually. Following the pregnancy the maternity belt supports the back formation of the abdomen tissue.

Indication

- abdominal relief
- support of spinal column
- prophylaxis of herniated nucleus pulposus
- decrease of pain

Instructions

Place the maternity belt at the area of the vertebrae lumbales and close beneath the abdomen and close with Velcro closure. Both straps are attached to the support beneath the abdomen as well.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopedic specialist or pharmacist to ensure the correct fitting of the ParadiSol 800. Do not wear the support while sleeping. Do not use ointment or creams while wearing the support.

Washing instruction (🧼 🚿 🌞 🧴 🚫)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

white