Thorax 910 (Damen)

Kostenlose Service Hotline

0800-ORPEDO.
0800-6773361

Orpedo
Orthopedics Company KG

Knesebeckstraße 54
D-10719 Berlin

Postfach 15 05 44
D-10667 Berlin

Tel +49-30-88 00 77 10
Fax +49-30-88 00 77 14

Web www.orpedo.com
Mail info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Specification

The Thorax 910 (Woman) reduces the movement of breathing of the chest by circular compression in the area of the thoracic cage. The healing of contusion and other traumas is supported.

Indication

- distortion of thoracic vertebral column
- rib contusion
- rib fracture
- inflammation of ribs and spine
- degenerative deformans
- postoperative

Instructions

Place the support round the ribs and close in front with Velcros.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Thorax 910 (Woman). Do not wear the support while sleeping. Do not use ointment or creams while wearing the support.

Washing instruction (☞ ☒ ☒ ☒ ☒)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

white