Orpedo

created for good life

Genuvit-Therm 500

Orpedo Germany GmbH

Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

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Status: 02/2017

Specification

The knee warmer Genuvit-Therm 500 made of soft, elastic and eudermic material protects against coldness and draft. The agreeable warmth relieves pain in the knee area. Due to the soft material and the good fitting the Genuvit-Therm 500 grants mobility and edges do not incise.

Indication

- slight knee pain caused by injuries or overuse or arthrosis
- rheumatic diseases
- gonalgia
- warming the knee

Contraindication

Unknown.

Ideal for allergic persons

Since no lamb's wool or angora is used.

Instructions

Pull the knee warmer the over afflicted part.

For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit-Therm 500.

Washing instruction (30°)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

wool white