Genuvit Comfort 310

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

Product description

The Genuvit comfort 310 offers an extension or flexion of the knee joint with the help of the 4-point-principle. Due to the two Velcro above and below the knee joint there will be taken individual consideration to the anatomy of the injured leg. The skin friendly material offers a high wearing comfort. With the help of the double height adjustment the orthoses can be adjusted to the leg length of the patient. The easy and toolless attitude feature takes place with moving to the desired angular degree. Afterwards the adjustment latches the protection. Genuvit comfort 310 is wearable left and right and reduces due to innovations and universal effort the attitude.

Limitation of the Extension: -15°, -0°, 15°, 30°, 45°, 60°

Limitation of the Flexion: 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°

Indikation

- rupture of anterior or posterior cruciate ligament
- fractur of patella
- gonarthrose
- pain reduction
- Injury of the inner and outer band
- postoperative
- post traumatic immobilization

Contraindication:

- dermal injury
- open wounds
- swellings
- allergies
- nerv irritation

Instruction

To properly wear the knee orthoses Genuvit comfort 310 it is recommended to sit skewed on the chair edge so the injured leg is free and can be extended. The Velcro of the Ortheses will be opened and pulled over the injured leg from behind. To ensure the correct fit of the product, it is recommended to close the Velcro below the knee. Afterwards the Velcro above the knee will be closed. The limitation of the Extension/the flexion has to be adjusted by a doctor or orthopaedic specialist.

Notice

The strapping closure of the orthosis is important for a good positioning. But there must not arise numbness. If the foot will become cold or you feel numbness, this can be caused by the closely adjustment or of the not correctly splint direction. In these cases the fitting has to be checked by a specialist and the splints have to be eventually adapted. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit comfort 310. Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication.

Material

inner:	outer:	metal portion:
85% Cotton	100% Polyamide	100% aluminium
15% Polyester		

Color
black

Washing instruction

The support product may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach.