
## Genuvit Comp 650

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

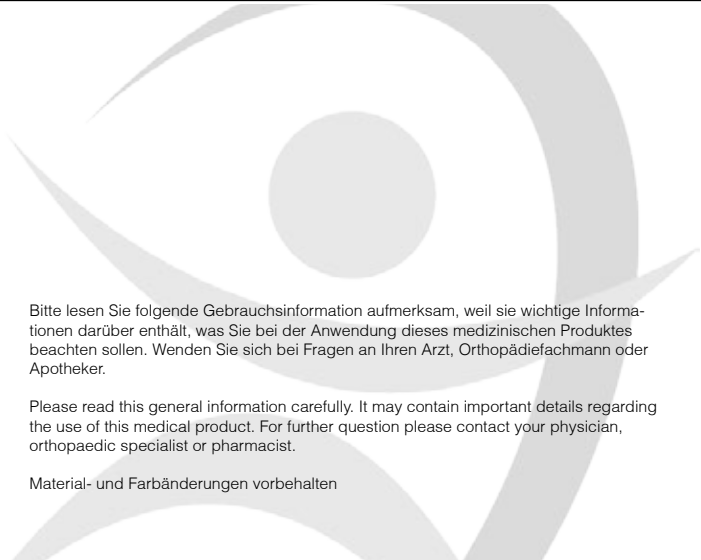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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Product description

The knee bandage Genuvit Comp 650, consisting of dual pull elastic material, is anatomi- cally knitted, has incorporated spirals along the sides, the patella area is closed and fitted with a silicone ring insole pad to lay the patella. A good fit, the special knitted comfort zone in the area of the popliteal cavity, spirals along the sides and the insole pad to lay patella guarantee hold and support the normal motions of the knee with an improved propriocep- tion. The breathable material is particularly kind to the skin, avoids the hypothermia of the knee and ensures a comfortable wearing.

## Indications

- irritations after operation, trauma, overburden of tendons, ligaments (lateral resp. medial) and/or muscular system
- chronic pain in the articulation of the knee caused by degenerative or rheumatic disease
- postoperative treatment
- slighty knee injuries (contusions, disortions, distensions)
- retropatellar chondropathy
- to stabilize the patella at lateralization

## Contraindication

Unknown.

## Instructions

Take the bandage at the upper ends of the spirals and pull it over the knee. The silicone ring must tightly enclose the patella. Because of the movement of the soft pads, a recor- rection might be necessary. Use the product according to the indications and the purpose.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit Comp 650 sw. In case of numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist to check the fit and adjustment.

## Washing instruction (☹️🚫🧼🚫🧼)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Color

black-white, skin