
## Genuvit fix 300

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

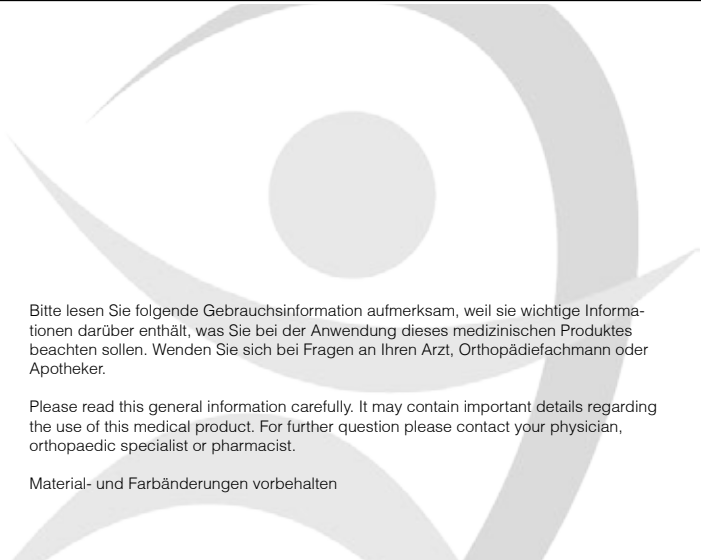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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Product description

By Genuvit fix 300 of Orpedo, the knee achieves the anatomical principle position. The anatomically well preformed braces and the Velcro fasteners being closely near the knee make an optimal position of the knee possible. The stabilizing rods can be taken out and adjusted according to the individual circumstances. The material is kind to the skin, washable at 30°C and allows wearing it in a hygienic and comfortable manner. The brace to immobilize the knee is suited to treat injuries of intervertebral ligaments and knee caps, before the operation for the detumescence and pain relief and after operations for the immobilization.

## Indications

- insufficiency of the capsule/ligament apparatus after trauma
- immobilization of the articulation of the knee after trauma / or immobilization before operations
- for postoperative care of fractures
- for post-traumatically or postoperative immobilization
- distortion, luxation in the articulation of the knee
- traumatic lesions, bursitis

## Contraindication

Unknown.

## Instructions

To put orthosis Genuvit fix 300 on the knee we recommend to sit down slanted on the edge of the chair so that the affected leg is stretched and free. Pull the orthosis from behind over the leg. To guarantee a correct fit of the orthosis, first close the Velcro fastener over the calf. Afterwards, close the other Velcro fasteners. The patella should lay in the opening of the orthosis. Use the product according to the indications and to the purpose.

## Notice

The strapping closure of the orthosis is important for a good positioning. But there must not arise numbness. If the foot will become cold or you feel numbness, this can be caused of the closely adjustment or of the not correctly splint direction. In these cases the fitting has to be checked by a specialist and the splints have to be eventually adapted. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Genuvit fix 300. Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication.

## Washing instruction (🧼🚰🚰🚰)

The support may be hand washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Color

black

## Material

Cotton  
Nylon  
Polyurethan sponge  
Polypropylene  
Polyester