## Humerovit 370

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Product description

The shoulder-arm orthosis offers a relief of the shoulder, upper arm and the elbow. Due to the completely skin friendly Material an ideal adjustment of the arm position is guaranteed. The position of the arm can be individually adjusted by the loop which is above the shoulder belt. The orthosis can be used for the left and right arm. The product is universal applicable for every arm length which contributes the ideal position. For children in the age of 4-9 there is a smaller version of the Humerovit 370.

## Indications:

- Injury of the rotator cuff
- distortion/ dislocation
- bruise
- humerus fracture
- posttraumatic/postoperative immobilization

## Contraindication:

- dermal injury
- open wounds
- swellings
- allergies
- nerv irritation

## Instruction

The name sling will be led backwards above the left or right shoulder, which depends on which arm is injured. The injured arm will be embedded in the bag which is made for it. Afterwards ground the arm with Velcro.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Humerovit 370.

## Material

inner:  
100% Polyester  
outer:  
100% Sponge

## Washing instruction (☒☒☒☒)

The orthosis may be washed by hand in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Color

black